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## Visual-Tactile Breast Examination Simulator

This simulator provides training in Visual-Tactile Breast Examination skills.

Four typical breast lumps can be differentiated by palpation.

The simulator is designed for medical education. Do not use for improper purpose.

Please read the instruction carefully before use.

### ● Features

- Four typical breast lumps are palpable.
- Examination skills for a case with skin dimple can be practiced.
- Collarbone and ribs are palpable.

### ● Set includes

Life size left breast model ..... 1

Talcum powder ..... 1

Size: W 200 x D 280 x H150 mm

### ⚠ DOs and DON'Ts

#### DOs

- Handle the model with utmost care at all times. Model skin is special composition of soft resins.
- Put talcum powder on the model skin before and after sessions. Add some powder while training sessions when the skin surface gets sticky.
- Clean the model by wiping soft cloth with water or soap water, dry naturally and put talcum powder before storage.
- For storage, lay the model horizontally, so that the breast faces upward.
- Do not leave the model it upright or up-side down for long time.
- To take off the case lid, open it and slide it horizontally to the left.

#### DON'Ts


- Do not pinch, dig your nail to the model or touch it by sharp instruments or handle it by excessive force. It may cause breakage in the skin, and replacement of the torn skin will be charged.
- Do not try to detach the model from the case. The model is fixed.
- Don't mark on the model with pen or leave any printed materials in contact with their surface. Ink marks on the models won't be removable.
- Never wipe the model with paint thinner or other organic solvent.

A) Cancer with a dimple sign

B) Cancer with a depression in the skin

C) Fibroadenoma

D) Mastopathy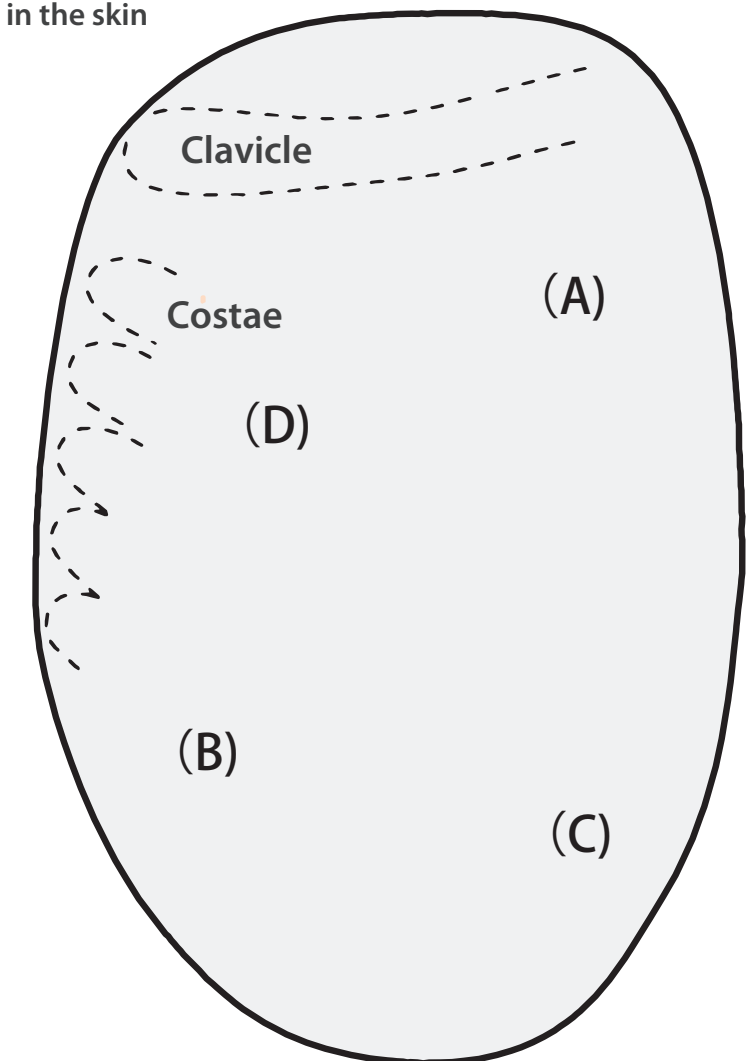

### Points to pay attention

- A) A dimple sign can be observed by putting the thumb and index finger on the surface with narrow (approx.3cm) gap, and bring the fingers closer to each other. The lesion is hard, and its surface is uneven. (Partially unclear boundary)
- B) The lesion is hard, and its surface is uneven. (Partially unclear boundary)
- C) A spherical lesion with smooth surface. Movable. (Clear boundary)
- D) A lesion with elasticity, smooth surface and unclear boundary.

 **KYOTO KAGAKU co.,LTD**

URL • <http://www.kyotokagaku.com> e-mail • [rw-kyoto@kyotokagaku.co.jp](mailto:rw-kyoto@kyotokagaku.co.jp)

#### ■ Main Office and Factory

15 Kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan  
Telephone : 81-75-605-2510  
Facsimile : 81-75-605-2519

#### ■ LA Office ( for USA , CANADA and Mexico customers )

3109 Lomita Boulevard, Torrance, CA 90505 ,USA  
Telephone : 1-310-325-8860  
Facsimile : 1-310-325-8867