

# Ear Examination Simulator

---

耳の診察シミュレータ

Instruction Manual	ENGLISH P.1~P.3
--------------------	-----------------

取扱説明書	日本語 P.5~P.7
-------	-------------

*KYOTOKAGAKU CO.,LTD. KYOTOKAGAKU CO.,LTD. KYOTOKAGAKU CO.,LTD.*

## Index

Features .....	P. 1
Contents .....	P. 2
Caution .....	P. 2
Practice .....	P. 3

## Features

**Ear Examination Simulator** is designed to help you practice an examination of the external ear and the eardrum with an otoscope. In addition, you can practice a removal of earwax.

**Ear Examination Simulator** will help you improve the skill of an examination.

- ① The external ear is closely reproduced. You can examine the external ear and the eardrum with an otoscope in your possession or available on the market.
- ② Two types of external ear with the normal eardrum are included. One is a beginner type and the other is an advanced type.
- ③ 8 kinds of cases are included in addition to the normal eardrum.
- ④ Their soft and supple material allows you to simulate a real examination, in ways such as pulling up the pinna.
- ⑤ Simulated ear wax and small toys are included for training in the removal of foreign bodies. An Ear without the eardrum is also included for this training.
- ⑥ The neck is flexible, which allows you to tilt and pose the head naturally, as in actual examination.

## Contents



① Mannequin

② Ears: 22 (11 matching sets of right and left)

1. normal	7. cholesteatoma
2. normal(bigger external auditory canal)	8. tympanosclerosis
3. serous otitis media;SOM	9. traumatic perforations
4. mucoid otitis media;MOM	10. cerumen block
5. chronic suppurative otitis media;CSOM	11. Practice of foreign substance removal
6. acute suppurative otitis media;AOM	

③ Simulated ear wax, small toy and small sponge sphere for a training at removing foreign bodies

④ Ear spoon

⑤ Powder

## Caution

- In order to simulate the patient, the head and ears of this model are made of soft material. So please do not handle the head and ears roughly.
- Please do not insert the otoscope too deep, as this may cause damage to eardrum.
- Marks left by ball-point or felt-tip pen are usually impossible to remove. So please do not make any marks unless you want to leave them permanently.
- For cleaning, wipe the mannequin with a soft cloth which has been moistened with water or alcohol. After drying the mannequin, please dust the head with baby powder.

## Practice



Choose an ear that has a case you plan to study. Insert the ear into the opening as shown in the picture. Putting powder on the opening makes insertion easier. To detach the ear, please grasp gently around the base of the pinna and pull straight out, perpendicularly to the head.



It is possible to tilt the head of the model as shown in the picture.



When you train in the removal of foreign bodies from the external ear, use ear No. 11.

After training, straighten the neck, and put **Ear Examination Simulator** back in the storage box.

For enquires and service, please contact your distributor or:

## **KYOTO KAGAKU CO., LTD.**

15 Kitanechoya-cho Fushimi-ku Kyoto, Japan 612-8388

Call 81-75-605-2510 Fax 81-75-605-2519

URL <http://www.kyotokagaku.com>

E-mail [rw-kyoto@kyotokagaku.co.jp](mailto:rw-kyoto@kyotokagaku.co.jp)