

耳の診察シミュレータ"EAR II" 簡単マニュアル

EAR Examination / 耳の診察		Use the "Exam Ears" (normal or stenosis) 診察用外耳 (通常・狭窄) を使用
NO	Cases	症例
1	normal	正常鼓膜
2	serious otitis media(SOM)	中耳炎 (漿液性)
3	mucoïd otitis media(MOM)	中耳炎 (粘液性)
4	chronic otitis media with perforation	中耳炎 (慢性穿孔性)
5	acute suppurative otitis media(AOM)	中耳炎 (急性化膿性)
6	cholesteatoma	真珠腫
7	tympanosclerosis	鼓室硬化症
8	traumatic perforations	外傷性鼓膜穿孔
9	cerumen block	耳垢閉塞
FB	foreign body removal (no eardrum image)	耳垢・異物除去

Foreign Body Removal / 耳垢・異物除去 Use the "FB Ears" (normal or stenosis)
異物除去用外耳 (通常・狭窄) を使用

Operation panel / 操作部

The diagram illustrates the operation panel with the following components and labels:

- Case number 症例番号**: A black box with the number '1'.
- External auditory canal sensor 骨部外耳道センサー**: A small grey circle.
- Eardrum sensor 鼓膜センサー**: A red circle.
- Case switch dial 症例切り替えダイヤル**: A large grey dial.
- Power 電源**: A switch with 'ON' (indicated by a vertical bar) and 'OFF' (indicated by a circle) positions.
- Mode change 症例切り替え**: A switch with two positions: 'Self-learning mode' (indicated by a circle) and 'Instructor/Examination mode' (indicated by a circle).
- Battery box / 電池ボックス**: A large rectangular box at the bottom.

KYOTO KAGAKU co.,LTD