

 **Caution** | Don't mark on the model and other components with pen or leave printed materials contacted on their surface.
Ink marks on the models will be irremovable.

M85 Airway Suction Trainer

Instruction Manual

Contents



Introduction	P.2
Manufacturer's Note / Dos and Don'ts	
Before You Start	P.3
Set Includes	
Before Training	P.4
Preparation	
After Training	P.5~P.7
Remove the skin on Airway Suction Trainer	
Wash the skin on Airway Suction Trainer	
Reinstall the skin on Airway Suction Trainer	

Movie Site



English Site

Manufacturer's Note

M85 Airway suction trainer is designed to facilitate understanding and training of suction catheter insertion and temporary airway suction.

Do not use the trainer for any other purpose.

Please read the instruction manual before use.

Product Features

1. Three approaches for suction catheter insertion: orally, nasally or through the tracheostomy opening .
2. Temporary airway suction by using simulated sputum.
3. Tracheostomy care and management.
4. Realistic life-size anatomy including mouth, tongue, oral and nasal pharynx, larynx, epiglottis, vocal cords, trachea, and esophagus (facilitates understanding insertion length of catheters).
5. An observation window allows the passage of the catheter to be demonstrated .
The tips of the provided catheters are colored to help the observation.
6. Trachea and bronchus can be easily flushed clean after use.
7. Face, neck, skin and airway unit can be removed for cleaning and is replaceable.



DOs and DON'Ts

DOs

- Wash the unit after each session.
- Powder the model skin with talcum powder before storage.
- Store the trainer in storage case at room temperature away from heat, moisture, and direct sunlight.

DON'Ts


- Do not ever wipe the simulator with thinner or organic solvent.
- Do not pull the skin-airway unit with force.
- Handle with care.
- Do not mark on the simulator with pen or leave any painted materials in contacted with the surface.
Ink marks on the product will not be removable.

Before you start

Set Includes

Set Includes

Before your first use, ensure that you have all components listed below.

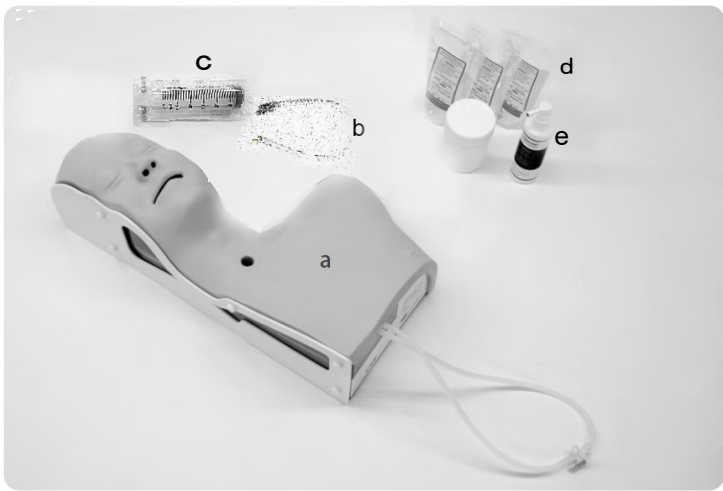

- a.Manikin (life-size, 459*230*150mm).... 1pc
- b.Catheter (14Fr).....2pcs
- c.Syringe (50ml).....1pc
- d.Simulated sputum(150ml×3/1jar).....1set
- e.Lubricant.....1pc

Consumables and replacement parts

Product Code	Part Name
11229-070	Simulated sputum/simulated watery stool
11229-020	Replacement skin with trachea for M85 1 piece
11229-040	Catheter for M85 2 piece (14Fr)
11229-050	A bottle of lubricant for M85 1 piece



Simulated sputum



Replacement skin with trachea



Catheter



Lubricant

Please note: It is recommended to fill the simulated sputum on a tray to avoid any stain from the liquid.

1 Preparation

1. Open the simulated sputum bag from the notch to open it.





2. Pour the simulated sputum to the accessory bottle.



3. Fill the syringe with the simulated sputum.



4. Connect the syringe tip into the three-way tap on the tube from the manikin and inject the simulated sputum into trachea or pharyngeal region.

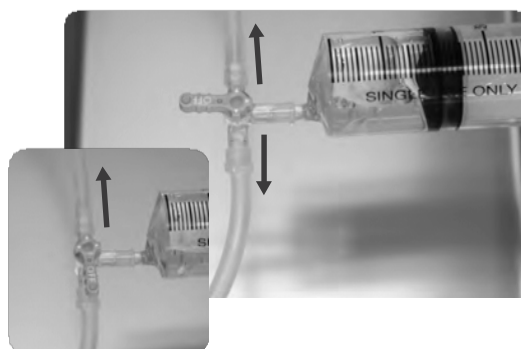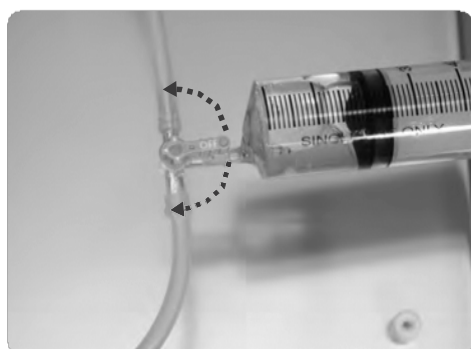

5. When the simulated sputum is:


Sticky	Add a little water and stir with stick.
Thin	Stir with stick so as to uniformize the viscosity.

6. Spray the lubricant in the nasal, oral, and tracheostomy openings to facilitate the smooth insertion of catheters.




1 Remove the skin of Airway Suction Trainer


1. Close the three-way tap.

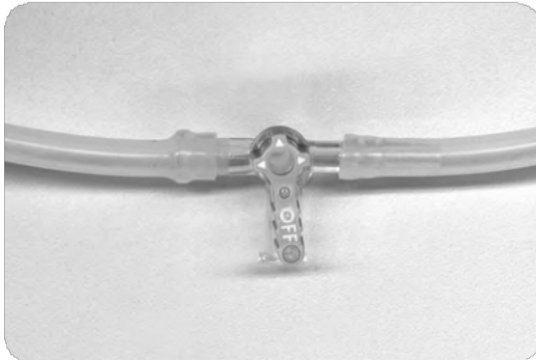

2. Detach the side plate by removing the screws.

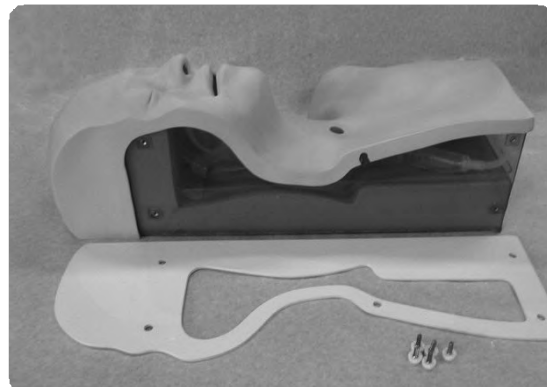

3. Remove the skin on Airway Suction Trainer from the base. Be cautious not to spill the liquid from the openings. If the liquid is spilled inside of the manikin, remove the transparent tray from the base and wash it with water.

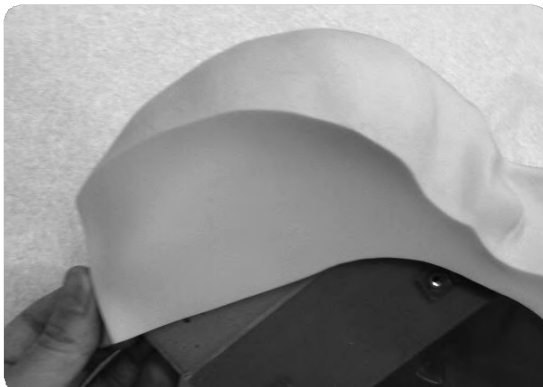

To remove the skin on Airway Suction Trainer , remove skin from the head side of the model.

2 Wash the skin on Airway Suction Trainer

1. Open the three-way tap.
2. Inject the water into the tube by the syringe to eject the simulated sputum.



3. Flush out the inside of the airway unit with running water through each opening.
Please note: It is recommended to wash the skin after each session.
This is recommended to avoid drying of the simulated sputum that may create blockage in the tube.



For easy water discharge, detach one of the tube tips from the three way tap.

4. Dry the unit thoroughly. Avoid direct sunlight.

3 Reinstall the skin on Airway Suction Trainer

1. Make sure that the nasal tubes are properly placed with the shape of the base.




3 Re-install the Skin on Airway Suction


2. Set the end of nasal/ trachea tube to the outside of the manikin through the notch on the transparent plastic wall.

Make sure not to tuck the tube between the skin and the base.

Place the end of the trachea unit along the inner side of the transparent plastic wall.

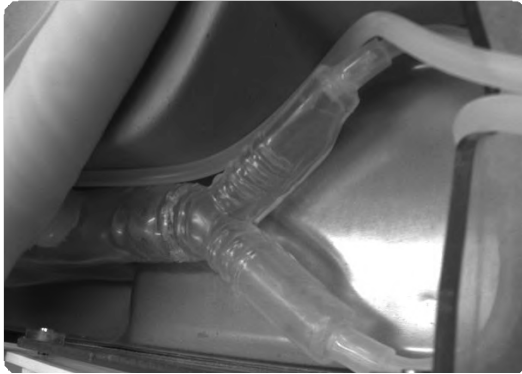

3. Fasten the skin with Velcro fasteners at the bottom of the manikin.

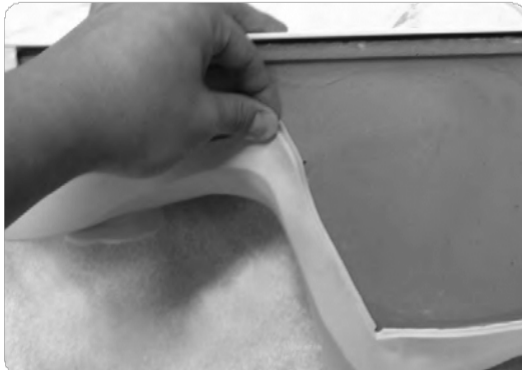

4. Cover the edge of the transparent wall with the skin.



5. Fasten the plastic side plate with screws.

Please note: Do not to fasten the screws too tight. Excessive force may cause breakage in the model.





Caution

Do not let ink from pens, newspapers, product manual or other sources contact the manikin. Ink marks on the manikin will be irremovable.

- For inquiries and service, please contact your distributor or KYOTO KAGAKU CO., LTD.



KYOTO KAGAKU co.,LTD

<http://www.kyotokagaku.com> e-mail: rw-kyoto@kyotokagaku.co.jp

■ Main Office and Factory

15 Kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan

Telephone : 81-75-605-2510

Facsimile : 81-75-605-2519

■ "Worldwide Inquiries and Ordering"

Asia and other regions

Kyoto Kagaku / Overseas Sales Division

Second fl. NREG Hongo-3-chome bldg., 3-26-6 Hongo, Bunkyo-ku, Tokyo 113-0033, JAPAN

■ All America regions

3109 Lomita Boulevard, Torrance, CA 90505-5108, USA

TEL : 1-310-325-8860 FAX : 1-310-325-8867

■ Europe, Russia & Africa

Kyoto Kagaku Europe GmbH

De-Saint-Exupery-Str.10, 60549 Frankfurt, Germany Tel: +49-69-5060-28160

The contents of the instruction manual are subject to change without prior notice.

No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer.

Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.

Please contact manufacturer with any discrepancies in this manual or product feedback. Your cooperation is greatly appreciated.