



Caution

Do not mark on the model and other components with pen nor leave printed materials in contact with surface of trainer.

MW30

11412-000

Head and Neck Palpation Trainer

MW31

11412-100

Head and Neck Palpation Trainer for Training and OSCE

Product Supervision :
Junji Otaki,
Honorary professor, Hokkaido University

Instruction Manual



Contents

- Before you start
Set includes/Dos and Don' ts P. 1
- Handling and Storage P.2
- Preparation
Case opening and identification of manikins
Pre practice procedures P.3
- Training
Regions for Palpation P.4
Creating swallow movement P.5
- After Training P.5

Set includes


Before your first use, ensure that you have all components listed below.

- Head and Neck Palpation Trainer



- | | | |
|---|-----------------------|---|
| a | Head and neck manikin | 1 |
| b | Talcum Powder | 1 |

- Head and Neck Palpation Trainer for training and OSCE

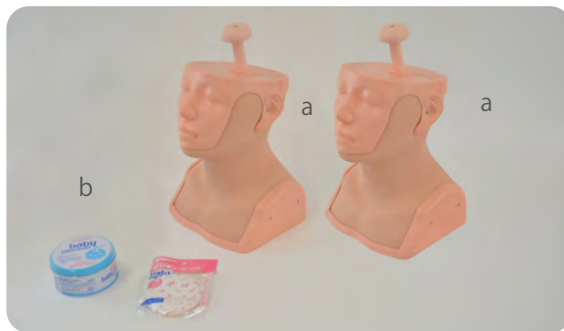

- | | | |
|---|---|---|
| a | Head and neck manikin (type A + type B) | 2 |
| b | Talcum Powder | 1 |
| | Instruction manual | |

Dos and Don'ts

<ul style="list-style-type: none"> ● Handle with care Materials of model are special compositions of soft resin, and carry a certain amount of weight. 	<ul style="list-style-type: none"> ● Wipe model with water or detergent Wipe stain or dirt with water or detergent. Do not use thinner or any liquid containing organic solvent.
<ul style="list-style-type: none"> ● Do not place any printed material on surface of model Prevent any other printed material from touching the surface of model. 	<ul style="list-style-type: none"> ● Do not use pen or ink on model When used on model, pen or ink will be absorbed by material and can not be removed.
<ul style="list-style-type: none"> ● Store model in a cool, dry place Prevent model from heat, humidity, and direct sunlight when storing after use. This is to prevent material change and damage. 	<ul style="list-style-type: none"> ● Do not store other printed material or item together with model Long period of pressure on material could cause material to change shape.
<ul style="list-style-type: none"> ● Do not lay flat during storage Make sure to store model with case in upright position. 	<ul style="list-style-type: none"> ● There could be discoloration in material Discoloration happens naturally when model is left unused for a certain amount of time. This will not affect the use of model.
<ul style="list-style-type: none"> ● Do not pinch and pull only the protective sheet Do not pinch and pull the protective sheet on the mannequin surface. 	<ul style="list-style-type: none"> ● Do not touch the manikin with sharp objects. Long periods of pressure on material could cause material to change shape.
<ul style="list-style-type: none"> ● Do not let model be in contact with other models during storage Being in contact with other models could cause material to react and change in quality. Always store in case after use. 	

PLEASE READ:

Always follow the handling and storage instruction below, to keep the material with its elasticity and durability.

○ Before and after use, apply talcum powder to the surface of the manikin.

To keep good condition of the simulator, apply talcum powder to the palpation area.

○ Store the manikin in the dedicated case for the simulator when not in use.







To prevent staining, deformation, or discoloration, store the manikin in the dedicated case for the simulator after use and when not in use.

○ Maintain designated temperature, humidity and avoid direct sunlight for storage

Store the simulator below 60 degrees celcius.

Change the storage place immediately when the temperature indicator shows color circles as below.

*The color circles are irreversible

Before			After		
3F60	年	月貼付	3F60	年	月貼付
					
60	65	70	60	65	70

○ Do not store other printed material or item together with model.

Long period of pressure on material could cause material to change shape.

1 How to open case

- Open storage case and take manikin.

Hold the bottom of the model and remove it from the box.



Note : Do not lift manikin with the palpation lever. It may cause malfunction.




- Identification of manikins

Check the bottom of the manikin for a sticker indicating the type.



Head and Neck Palpation Trainer (T Type)

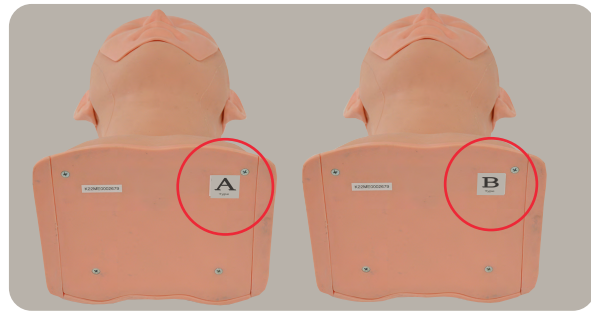

Head and Neck Palpation Trainer for training and OSCE (A Type、 B Type)

2 Pre-practice procedures

- To facilitate palpation, apply the supplied talcum powder to the palpation site on the manikin.



If friction is felt at the palpation site, apply additional talcum powder to the surface of the manikin. Applying powder to the fingers is also effective.

1 Regions for Palpation


● Head and Neck Palpation Trainer

Palpation of cervical lymph nodes
11 place


Examination of thyroid gland
1 place

T Type

Right side

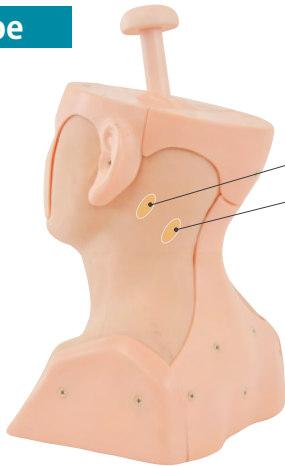

posterior auricular nodes
Occipital nodes
Superficial cervical nodes
Posterior cervical nodes
Deep cervical nodes
Sternocleidomastoid

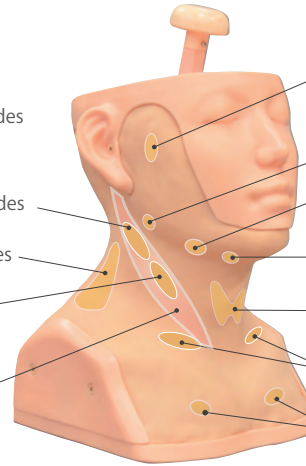

Pre-auricular nodes
Tonsillar nodes
Submandibular nodes
Submental nodes
Thyroid gland
Supraclavicular nodes
Subclavicular (Infraclavicular) nodes


Left side

● Head and Neck Palpation Trainer for training and OSCE


A Type

Right side

Submandibular nodes
Posterior cervical nodes
Deep cervical nodes

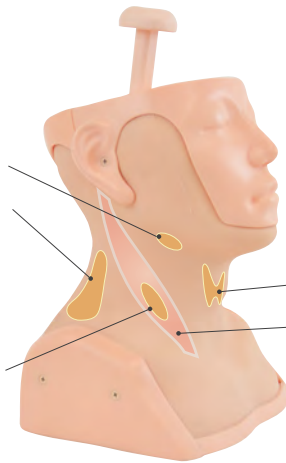

Pre-auricular nodes
posterior auricular nodes
Superficial cervical nodes
Thyroid gland
Sternocleidomastoid
Supraclavicular nodes

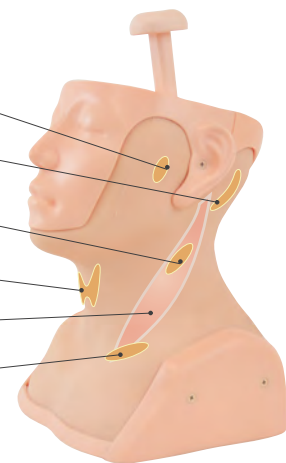


Left side


B Type

Right side

Submandibular nodes
Deep cervical nodes
Subclavicular (Infraclavicular) nodes

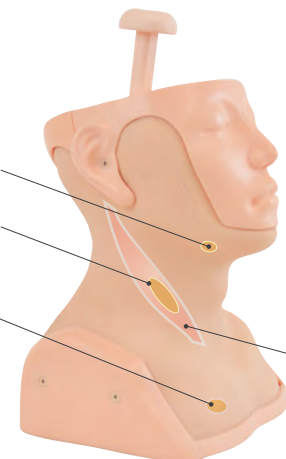

Superficial cervical nodes
Submental nodes
Submandibular nodes
Sternocleidomastoid
Posterior cervical nodes

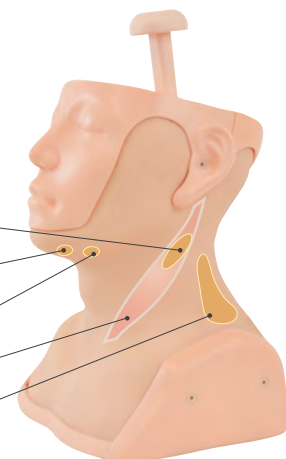

Left side



Caution

Do not let ink from pens, newspapers, product manual or other sources contact the manikin. Ink marks on the manikin will be irremovable.

- For inquiries and service, please contact your distributor or KYOTO KAGAKU CO., LTD.

■ Main Office and Factory (World Wide)



KYOTO KAGAKU co.,LTD

WEB • www.kyotogagaku.com

E-MAIL • rw-kyoto@kyotokagaku.co.jp

15 kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan

TEL: +81-75-605-2510

FAX: + 81-75-605-2519

■ North and South American regions:



KKAmerica

Kyoto Kagaku America Inc.

WEB • www.kkamerica-inc.com

E-MAIL • info@kkamerica-inc.com

3109 Lomita Boulevard, Torrance, CA 90505-5108, USA

TEL: +1-310-325-8860

(Toll-free in North America: 877-648-8195)

FAX: +1-310-325-8867

The contents of the instruction manual are subject to change without prior notice.
No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer.
Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.
Please contact manufacturer with any discrepancies in this manual or product feedback. Your cooperation is greatly appreciated.