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**MW54 Strap-on Female Catheterization / Enema Simulator**

**MW55 Strap-on Male/Female Catheterization / Enema Simulator**

Thank you for choosing our products. Please perform the following maintenance to keep good condition and for a long time.

- Lubricate the urethra and the valve with the included maintenance catheter before every training session. After lubrication, insert the maintenance catheter rotating from side to side to lubricate the parts adequately.

\* Inadequate lubrication may cause the damage of the products. Lubricate the parts adequately with the maintenance catheter.

- Apply a little more lubricant to MW53 (Male Model) as the urethra of Male is longer than Female Model as same as human.

\* Inadequate lubrication may cause difficulty of catheter insertion. In case apply lubricant to catheter adequately once again.

- Appropriate catheter is as follows.  
Male (MW53) 16Fr : Female (MW54) 14Fr

\* Use of larger size catheter may cause damage of the valve earlier than usual.

- Do not use a lubricant other than included.

\* As Lubricity of Jelly type lubricant (water soluble type) is lost in water, the valve is damaged earlier than usual.

\* Please access our web site via following QR code if you have a question about how to use of lubricant.

<https://www.kyotokagaku.com/jp/video/mw55.html>

## WORLDWIDE INQUIRIES & ORDERING

 **KYOTO KAGAKU CO.,LTD**

<http://www.kyotokagaku.com>  
[rw-kyoto@kyotokagaku.co.jp](mailto:rw-kyoto@kyotokagaku.co.jp)

### ● KYOTO KAGAKU HEAD OFFICE & FACTORIES

15 Kitane-koya-cho, Fushimi-ku, Kyoto 612-8388,  
JAPAN

TEL: +81-75-605-2510 / FAX: +81-75-605-2519

## ALL AMERICAN REGIONS

 **KYOTOKAGAKU AMERICA INC.**

<http://www.kkamerica-inc.com/>  
[info@kkamerica-inc.com](mailto:info@kkamerica-inc.com)

3109 Lomita Boulevard, Torrance, CA 90505-5108,  
USA

TEL: +1-310-325-8860 / FAX: +1-310-325-8867