

11489-100

MW85

Clinical Assessment Kit

- Cyanosis -



Caution

This product is NOT compatible with
MW22 Patient Care Simulator "Cherry"

Instruction
Manual

Recommended simulators to use with

- MW25 Patient Care Simulator "Yaye"
- MW26 Basic Patient Care Simulator "Keiko"



Precautions

● **Do not use this product for purposes other than its intended use**

The product is designed for training of healthcare professionals. Usage for any other purpose or the way of use not described in this manual is strongly discouraged.

● **This product is to be used with Kyoto Kagaku Simulators and Mankins.**

The manufacturer cannot be held responsible for use with third-party products.

● **Don't use on human**

If the product comes into contact with your hands or skin and causes any abnormalities such as itching or redness, discontinue use immediately, rinse thoroughly with water, and consult a doctor if necessary.

● **Close the tube cap immediately after use.**

● **After use, wipe the cyanosis paint off the brush.**

Once the required amount of cyanosis paint has been squeezed out of the tube, close the cap immediately. After use, promptly wipe the paint off the brush. The paint may dry out and harden, making it unusable.

● **Always remove the cyanosis layer from the simulator after use.**

Be sure to remove the cyanosis layer after use. Leaving it on for an extended period may cause discoloration or surface deterioration of the simulator.

● **Store dark, cool and dry space will help prevent the product's colors from fading and product from deforming.**

■ Related product

11489-000 MW85 Clinical Assessment Kit -Surgical Wound -

Set Includes



- a. cyanosis tube (10ml).....4
b. painting brush..... 1
c. storage bag..... 4

*at the time of delivery "a" is packed in "c"



KYOTO KAGAKU co.,LTD

www.kyotokagaku.com rw-kyoto@kyotokagaku.co.jp

Kyotokagaku Head Office and Factories:

15 Kitanechoya-cho, Fushimi-ku, Kyoto, 612-8388, JAPAN, Tel: +81-75-605-2510, Fax: +81-75-605-2519

Kyoto Kagaku America Inc: All America regions

3109 Lomita Boulevard, Torrance, CA 90505-5108, USA,
info@kkamerica-inc.com Tel: 1-310-325-8860, Fax: 1-310-325-8867 (Toll free in North America: 877-648-8195)

Kyoto Kagaku Europe GmbH. Europe, Russia & Africa

De-Saint-Exupery-Str.10, 60549 Frankfurt, Germany, rw-kyoto@kyotokagaku.co.jp Tel: +49-69-5060-28160

Preparation / How to Apply

1. Clean the application surface.

Wipe off any oil or dust with alcohol.



2. Squeeze a small amount onto the brush

Open the storage bag and remove the tube.



Squeeze a small amount of the contents onto the brush.



3. Apply to the nails and/or lips of the simulator.

Gently apply the cyanosis paint to the target area as if placing a drop of water, then spread it slowly using the tip of the brush.

- The cyanosis layer dries within 1–5 minutes when using a dryer. Consider the curing time when planning the session. It is recommended to apply the cyanosis layer one day in advance.

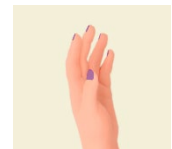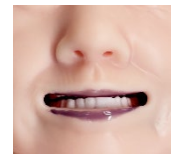

Tips

The key is to avoid pressing the brush during application. Pressing too firmly may leave brush marks and result in an unnatural appearance.

- ✗ Press the brush
- Put and spread gently

<https://youtu.be/Rhp2erpD6rg>

Tips for application in 120 seconds



YouTube

After Application

4. Always wipe the brush before the cyanosis paint dries out.

Thoroughly wipe the used brush with a soft cloth moistened with alcohol or water. Wipe the tip of the tube, close the cap firmly, and store it in the storage bag. Once opened, use the contents as soon as possible.



5. Always remove the cyanosis layer from the simulator after use.

Once cured, the cyanosis layer can easily be peeled off. Leaving it on for an extended period may cause discoloration or surface deterioration of the simulator.



If the paint gets on clothing

If the cyanosis paint gets on clothing, wash it off immediately with a mild detergent. Once it dries, removal may be difficult.

*If the product comes into contact with your hands or skin and causes any symptoms such as itching or redness, discontinue use immediately, rinse thoroughly with water, and consult a physician if necessary.